

## Flu Screen® Test Procedures




**Throat swab**



**Solution added**



**Test strip inserted**

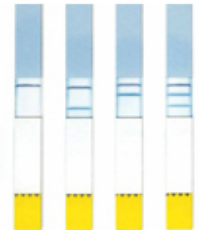

**Results**

### Flu Screen® procedures

- The flu screen test starts with a swab sample collection.
- The samples then need to be extracted using the extraction solution.
- After the 5 minute extraction step, the sample can be tested with the test strip and the results are available at 10 minutes after dipping.

### Results indicated

- A single blue line in the lower result area indicates a positive result for influenza type A,
- A blue line in the middle part indicates type B.
- If only the black control line is visible, the result is negative.